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Usado para entablillar el dedo(s)

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ID# *27444*

R404

EUROPEAN AUTHORIZED REPRESENTATIVE:
MDS&S GmbH
Schiffgraben 41
D-30175 Hannover Germany
Council Directive 93/42/EEC
of 14 June 1993 concerning Medical Devices

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PROFESSIONAL PRODUCTS, INC.
54 HUGH ADAMS ROAD
DEERBARK SPRINGS, FL 32435

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Fingerschiene

FOR OVER 50 YEARS Proudly MADE IN USA

English APPLICATION INSTRUCTIONS

INDICATIONS
Used to splint the finger(s)

Thoroughly read the following information prior to application.
Product function requires proper application.

A Place Finger
Splint on affected
finger(s):
Bend adjustable
aluminum stay
to the desired
contour of hand and
finger(s) (Fig. 1).

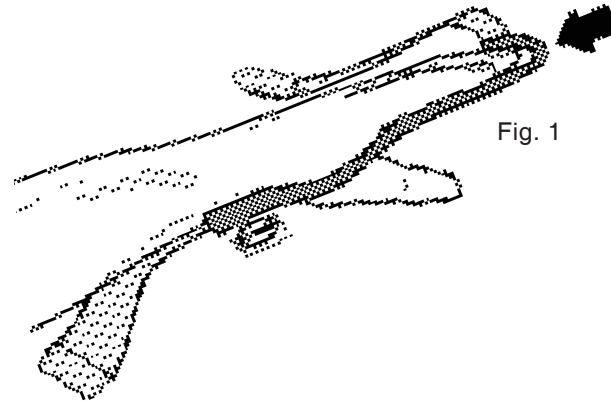
Fig. 1

B Secure Permanent
Straps in the order
shown:
(Fig. 2 # 1 & #2).

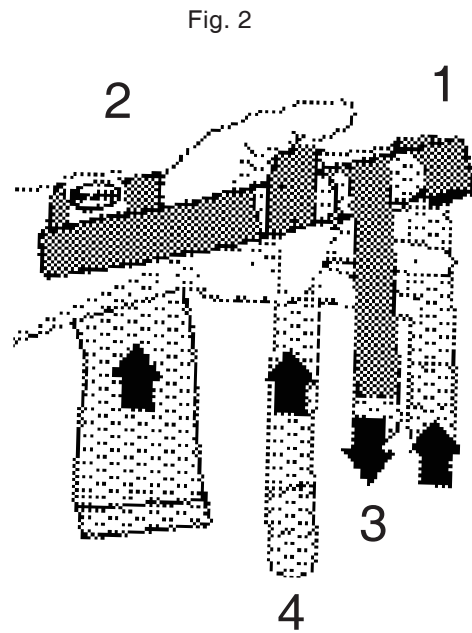
Fig. 2

C Position and
secure Removable
Straps: Secure
shorter strap (Fig.
2 #3) at base
of fingers, then
secure final strap
(Fig. 2 #4) around
palm of hand.
Figure 3 shows
one example of
the many possible
applications
available.

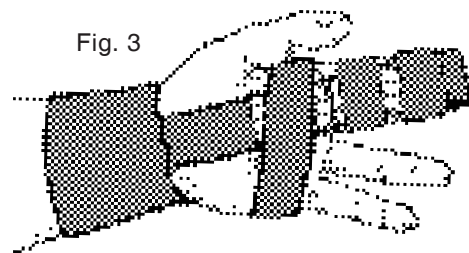
Fig. 3

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat

WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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